NUK[®]

JOLIE

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Contents

1. Important information before using for the first time	14
2. Pump description	14
2.1 Intended use	14
2.2 Description of function	14
2.3 Contra-indications	14
2.4 Side effects	14
2.5 Symbols on the pump/packaging	14
3. Safety information	15
3.1 Explanation of terms	15
3.2 General information	15
3.3 Hygiene	15
3.4 Notes for the Breast Milk Container and Teat	16
4. Key parts (Figure O)	16
5. Taking the pump apart (Figures A to F)	16
6. Cleaning and sterilising	16
6.1 Cleaning	16
6.2 Sterilising	17
7. Reassembling the pump (Figures G to N)	17
8. Using the breast pump	17
8.1 Expressing milk	17
8.2 Feeding expressed milk	18
9. Storing milk	18
9.1 NUK Breast Milk Container	18
9.2 Important information for handling breast milk	19
10. Tips for successful expressing	19
10.1 Frequency of expressing	19
10.2 Method	20
10.3 Quantity of milk	20
10.4 Let-down reflex	20
11. Troubleshooting	21
12. Technical data	21
13. Disposal	21
14. Warranty	21
14.1 Service Centres	22
14.2 Manufacturer	22
15. EU declaration of conformity	22

Dear Mum,

Breast milk is the best food for a baby during the first six months of its life. To enable you and your baby to enjoy every moment of breastfeeding, NUK has developed a range of practical and comfortable breastfeeding aids for the systematic support of breastfeeding.

The NUK JOLIE Manual Breast Pump is designed to help you and your baby whenever natural breastfeeding is not possible.

Before using your NUK JOLIE Manual Breast Pump for the first time, please take a few minutes of your time to read these instructions for use carefully.

1. Important information before using for the first time

Warning! Before using the NUK JOLIE Manual Breast Pump for the first time, clean and sterilise all parts except part **15** (see “6. Cleaning and sterilising” on page 16).

Please read carefully the enclosed information on use and warnings for the NUK breast milk container / bottle and teat, as well as the accessories.

2. Pump description

2.1 Intended use

The NUK JOLIE Manual Breast Pump is an aid to expressing and collecting breast milk:

- in the workplace, when baby is away, at home or when travelling
- to maintain milk production during a break in breastfeeding
- to relieve engorgement
- to increase milk production

Once expressed, breast milk can be collected, stored, frozen, warmed and fed to your baby using the NUK Breast Milk Container.

2.2 Description of function

The NUK JOLIE Manual Breast Pump is a manual suction pump and offers the following benefits:

- simple to use and needs no power supply
- breast pump comes apart for easy cleaning
- silicone cushion for gentle massaging effect and softer feel
- operating the pump imitates your baby's natural suckling action
- collects expressed breast milk

- supplied with stand so that it can be put down safely

2.3 Contra-indications

If you suffer from certain breast ailments (inflammation, open wounds), please consult your doctor, midwife or breastfeeding counsellor before using the NUK JOLIE Manual Breast Pump.

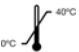
2.4 Side effects

If at any time you experience an unexpected reaction, please consult your midwife, health visitor, breastfeeding counsellor, doctor or pharmacist.

Please report any serious incident in relation to the device to the manufacturer and your national competent authority (see “14.2 Manufacturer” on page 22).

2.5 Symbols on the pump/ packaging

Symbol	Meaning
	Manufacturer
	Refer to the instructions for use

Symbol	Meaning
	Keep away from sunlight
	Temperature limit for storage +0 °C to +40 °C
	Article number
	Batch identification

Symbol	Meaning
	Attention! Observe the safety instructions in the instructions for use
	Declaration of conformity Complies with all the requirements of applicable EC directives
	Please note that the packaging can be recycled and dispose of it accordingly
	Medical device

3. Safety information

3.1 Explanation of terms

The following keywords are used in these instructions for use:

Warning!

Ignoring this warning could result in a serious risk to life and limb.

Caution!

Ignoring this warning can cause minor personal injury and / or material damage.

Note:

Points to be noted when handling the device.

3.2 General information

- Before using the pump, please read these instructions for use carefully. These instructions are part of the device and must be kept to hand at all times.
- Use the NUK Breast Pump and accessories only for their intended purpose (see “2.1 Intended use”)

Warning! To avoid risk of choking, keep small parts out of the reach of children.

- We recommend removing ointments/creams containing grease or oil from the breast before expressing, as these may damage the surface of the silicone cushion.
- To guarantee that the pump will work correctly, only use original NUK accessories.

- Please note that any claim under warranty or liability will be void if accessories are used which are not recommended in these instructions for use, or if any parts other than original spare parts are used for repairs. This also applies to repairs carried out by unqualified persons.
- In the event of any malfunctions, please contact the manufacturer.
- If the breast milk is for a premature baby, it is essential to follow your doctor's instructions.

3.3 Hygiene

Warning! When handling breast milk, hygiene is of particular importance to ensure your baby's healthy development.

- To prevent infection or bacterial contamination (microbial growth) and to prevent damage to the pump, please read and pay attention to “6. Cleaning and sterilising” on page 16.
- **Only a breast pump which has been thoroughly cleaned will achieve full suction strength.**
- Remove all body jewellery (e.g. piercings) before using the breast pump.

3.4 Notes for the Breast Milk Container and Teat

Please read carefully the enclosed information on use and warnings for the NUK breast milk container / bottle and teat, as well as the accessories.

For your child's safety and health: **WARNING!**

4. Key parts (Figure O)

- | | | | |
|---|-------------------|----|-------------------------------------|
| 1 | Silicone cushion | 9 | Lip valve |
| 2 | Breast shield | 10 | NUK Breast Milk Container |
| 3 | Slot for pump rod | 11 | Sealing disc |
| 4 | Pump lever | 12 | Teat |
| 5 | Handle fixture | 13 | Screw ring |
| 6 | Pump rod | 14 | Protective cap |
| 7 | Pump diaphragm | 15 | Stand for NUK Breast Milk Container |
| 8 | Catch for handle | | |

5. Taking the pump apart (Figures A to F)

1. Take the silicone cushion **1** out of the breast shield **2** (Fig. **A**).
2. Holding the end of the handle, carefully pull the pump lever **4** straight upwards, until the handle fixture **5** slips out of the catch for the handle **8** (Fig. **B**).
Attention! Don't pull/turn the pump lever sideways, otherwise the pump rod may break.
3. Carefully remove the pump lever **4** from the pump rod **6** (Fig. **B**).
4. Take the pump diaphragm **7** and pump rod **6** out of the breast shield **2** (Fig. **C**).
5. Ease the pump rod **6** out of the pump diaphragm **7** (Fig. **D**).
6. Unscrew the NUK Breast Milk Container **10** from the breast shield **2** (Fig. **E**).
7. Pull the lip valve **9** out of the breast shield **2** (Fig. **F**).
8. Finally, clean the individual parts as described in "6. Cleaning and sterilising".

6. Cleaning and sterilising

Warning!

Hygiene is of the utmost priority when handling breast milk and all parts that come into contact with it. **The pump should therefore only be used in a clean and sterile condition.**

Clean, sterilise and quickly dry **all parts** before using for the first time and immediately **after each use**.

6.1 Cleaning

Caution! To avoid damaging the lip valve **9**, clean it carefully. We recommend just

rinsing off the lip valve and boiling it to sterilise. Cleaning implements could cause damage.

1. Clean all parts with warm water and a little washing-up liquid (or NUK Baby Bottle Cleanser).
2. Rinse off all parts in running water.

Warning! Milk residues may damage the material and result in microbial growth.

Caution! The parts should not be cleaned in a dishwasher – dishwasher detergents can cause material damage.

Note: When rinsing with dishes (for example), certain foodstuffs (e.g. carrot juice) may discolour the parts or affect their odour (e.g. fennel tea). This has no effect on the fitness for use or safety of the products.

3. Next, sterilise all components (see “6.2 Sterilising”).

6.2 Sterilising

Before sterilising, clean as described in “6.1 Cleaning”.

Caution! In order to prevent product damage, follow the manufacturer's instructions when using any sterilisers or sterilising solution.

1. Always dismantle the components before boiling or steam-sterilising (temperature

above 80 °C) and leave them dismantled, otherwise deformation may result.

2. Boil the parts for 5 minutes in a pan of vigorously boiling water.

Ensure there is sufficient water in the pan to avoid damage.

3. The sterilised components must be **thoroughly cooled down and completely dry** before being reassembled and used again.

Notes:

- Use a NUK Steam Steriliser. Follow the instructions for use associated with the device.
- For sterilisation in a microwave oven, we recommend using the NUK Microwave Steriliser (please follow the manufacturer's instructions for the steriliser/microwave oven!).

7. Reassembling the pump (Figures G to N)

1. Insert the pump rod **6** in the pump diaphragm **7**. The lower end of the pump rod **6** must fit in the hole of the pump diaphragm **7**. (Fig. **G**).
2. Position the pump diaphragm **7** in the breast shield **2** and press firmly in place (Fig. **H**).
3. Insert the head of the pump rod **6** in the pump lever **4** (Fig. **I**).
4. Engage the handle fixture **5** in the catch for the handle **8** (Fig. **K**).
5. Insert the lip valve **9** in the breast shield **2** (Fig. **L**).
6. Screw the NUK Breast Milk Container **10** onto the breast shield **2** (Fig. **M**).
7. Put the silicone cushion **1** in the front of the breast shield **2** and slip the edge of the cushion over the breast shield (Fig. **N**).

8. Using the breast pump

8.1 Expressing milk

Warning! As hygiene is so important, please wash your hands before expressing your milk, and read the information in “9. Storing milk” on page 18.

1. Place the funnel of the breast shield over your breast.

– Make sure that your nipple is in the centre of the silicone cushion **1**. This prevents the nipple becoming sore.

– Gently press the breast shield **2** against the breast so that the silicone cushion **1** fits closely and no air is drawn in from the side.

– **Always** hold the breast milk container and breast pump **upright!**

2. Press the pump lever **4** then slowly release it.

Note: The further you press the pump lever **4**, the greater the suction. To reduce the suction, do not press the pump lever **4** all the way.

Please pay attention to the information in “10. Tips for successful expressing”.

Caution!

- To achieve optimum results, express only enough milk into the container so that it does not come into contact with the bottom edge of lip valve **9**.
 - When freezing breast milk, do not fill the container with more than 150 ml (5 fl oz), as breast milk expands as it freezes and the container could burst.
3. After expressing, stand the breast pump upright. The pump must not be allowed to topple over, otherwise milk might escape.
 4. Unscrew the breast shield **2** from the breast milk container **10**.

5. Firmly seal the breast milk container **10** using the supplied screw ring **13** and sealing disc **11**.
6. Immediately after expressing, wash the individual parts (see “6. Cleaning and sterilising” on page 16).

8.2 Feeding expressed milk

Screw feeding teat **12** onto breast milk container **10** using screw ring **13**.

If the breast milk you have collected is not for immediate use, firmly seal the filled breast milk container **10** with the sealing disc **11** and screw ring **13** and store it in your refrigerator or freeze it (see “9. Storing milk” on page 18).

9. Storing milk

9.1 NUK Breast Milk Container

If the breast milk is not for immediate use, you must seal the filled breast milk container firmly and store it in the refrigerator or freeze it.

Warning! Do not place the entire NUK JOLIE Manual Breast Pump in the refrigerator, just the breast milk container **10** with screw ring **13** and sealing disc **11**.

Breast milk may be chilled, frozen and defrosted in the breast milk container. The container may also be used for warming and feeding. It is important that you follow the instructions about handling breast milk which is provided below.

Note: When freezing breast milk, do not fill the container with more than 150 ml (5 fl oz), as breast milk expands as it freezes and the container could burst.

9.2 Important information for handling breast milk

Breast milk	Storage location	Storage periods
Freshly expressed	At room temperature	Max. 3-4 hours
	Refrigerator (not in door)	72 hours at 4 °C
	Refrigerator freezer compartment	2 weeks
	Upright/chest freezer	6 months at -18 °C
Defrosted, unopened	Refrigerator (not in door)	24 hours at 4 °C
Defrosted, opened	Refrigerator (not in door)	12 hours at 4 °C
Defrosting milk	Gradually in refrigerator	Approx. 24 hours at 4 °C
	At room temperature followed by immediate feeding	
	If time is short, defrost container with breast milk in lukewarm water (max. 37°C), then feed immediately.	

- **Warning!** If transporting chilled breast milk make sure that it stays chilled at all times, otherwise the milk could go off and be harmful to your baby. Use a cooler bag with ice packs, for example.
- **Warning!** Breast milk must not be thawed in a microwave as this will destroy valuable nutrients as well as potentially causing damage to the container (possible risk of explosion). Uneven heating may result in scalding!
- To attain the correct feed temperature of approx. 37 °C, place the container of thawed breast milk in a NUK Baby Food Warmer.

Note: The container must be open to prevent pressure build-up as the milk warms up.
- Shake the milk gently before feeding so that milk and milk fat are properly mixed again.
- Defrosted, warmed or unused milk must be disposed of and not refrozen.
- If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to the previously cooled breast milk.

However freshly expressed milk must be cooled down before being added to milk that is already cool. **The temperatures must be the same (cold on cold).**
- Consult your health visitor, breastfeeding counsellor, midwife, doctor or pharmacist for detailed information about long-term storage.
- You can find more information about breastfeeding on our NUK website (www.nuk.com).

10. Tips for successful expressing

Correct expressing takes practice. You will first need to get used to your new pump.

10.1 Frequency of expressing

How often you need to express depends on the reason for expressing. If you are away from your child and/or want to sustain milk production, the natural breastfeeding

rhythm should be maintained. You should therefore express 6 to 8 times daily, including once during the night. In all other cases you should express entirely according to your personal wishes and requirements.

10.2 Method

Milk should be expressed from each breast in turn.

The amount of time needed to express milk will vary considerably from one person to another.

10.3 Quantity of milk

Don't be surprised if, at first, it takes a little while for the milk to start flowing properly. This is not unusual. In fact it is quite normal because **milk production adapts itself to daily requirements.**

This means that the quantity of milk produced may vary considerably.

- Always make sure that you drink plenty of liquids.

10.4 Let-down reflex

During natural breastfeeding, placing the baby on the breast stimulates the let-down reflex and milk begins to flow. You can also stimulate the let-down reflex as follows:

- Find yourself a quiet place to sit, where you can relax fully when expressing, and avoid stress.
- Adopt a comfortable and relaxed sitting position.
- Place a warm compress on your breast for a few minutes to relax the tissue, and massage your breast.
- Ensure you have eye contact with your baby. If this is not possible, it may help to look at a photo of your baby.
- The let-down reflex triggers milk flow in both breasts simultaneously. You can use the NUK Breast Shell Set to collect breast milk leaking from the breast you are not currently expressing.

11. Troubleshooting

Problem	Reason	Remedy
Pump has little or no suction	The pump rod 6 is not correctly located in the pump diaphragm 7	Make sure that the pump rod 6 is fully inserted in the hole in the pump diaphragm 7 (see “7. Reassembling the pump (Figures G to N)” on page 17).
	Pump diaphragm 7 missing, fitted incorrectly or damaged	Insert pump diaphragm 7 correctly (see “7. Reassembling the pump (Figures G to N)” on page 17). If anything is damaged, please contact the NUK Service Centre (see “Contact addresses” on page 87).
	Lip valve 9 is incorrectly positioned	Position lip valve 9 correctly (see “7. Reassembling the pump (Figures G to N)” on page 17).
	Lip valve 9 missing, fitted incorrectly or damaged	Check that the lip valve 9 is correctly positioned. If anything is damaged, please contact the NUK Service Centre (see “Contact addresses” on page 87).
	Breast milk container 10 is full and lip valve 9 is touching the expressed milk	Change the breast milk container 10 .
	No vacuum is being produced	Make sure that the breast shield 2 is making a complete seal around the breast.
		Make sure that the silicone cushion 1 is inserted correctly and is slipped over the edges of the breast shield.
Pump is being held at an angle	Hold the pump as upright as possible.	

12. Technical data

Medical device	Regulation (EU) 2017/745
Temperature range	
- Operation	+6 °C to +40 °C
- Storage	0 °C to +40 °C

13. Disposal

If at some stage the device is no longer fit for use, the parts can be disposed of in normal household waste.

14. Warranty

In addition to the seller's warranty obligations arising from the contract to purchase, we as the manufacturers provide a warranty of 24 months from purchase of the device as long as the device is used properly and the instructions for use are

followed. Evidence of the date of purchase and the type of device should be provided by means of a receipt.

The purchaser's statutory rights are unaffected by this manufacturer warranty.

During the warranty period, we undertake to remedy any defects attributable to poor materials or workmanship. Wear parts are excluded from the warranty.

Minor deviations from specified quality which do not affect the value or usability of the device are not covered by the warranty. We are likewise unable to accept the warranty if the defects on the device are due to transport damage for which we are not responsible, if they are attributable to misuse or lack of care or if the device has been interfered with by persons not authorized by us to do so.

The warranty service will involve repair, replacement of parts or replacement of the device at our discretion. The warranty period will be neither extended nor restarted by the performance of warranty services. The warranty for any spare parts fitted will expire with the warranty for the device as a whole.

Further or different claims, especially those relating to the making good of damage

sustained outside the device, are excluded unless liability is compulsory in law.

We do not accept the costs or risks of transport.

A device sent in without evidence of the purchase date will be treated as a repair. The device will only be repaired following discussion with the customer.

Retain address, article number and batch identification no. for possible future reference. The batch identification no. can be found on the breast shield (see Fig. O)

14.1 Service Centres

Addresses for service can be found under "Contact addresses" on page 87.

14.2 Manufacturer

Please note that the address below **is not a service address**. In the event of problems or queries relating to your product, contact one of the Service Centres (see "Contact addresses" on page 87).

MAPA GmbH
Industriestraße 21-25
27404 Zeven
Germany
www.nuk.com

15. EU declaration of confirmity

The device meets all applicable European directives, as well as their respective essential requirements. These are detailed in the EU declaration of conformity, which can be requested from the manufacturer.

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AUS - AUSTRALIA

Newell Australia Pty Ltd
Caribbean Park
Level 3, 35 Dalmore Drive
Scoresby VIC 3179
www.nuk.com.au

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Lapsekas Oy
Kaupinkatu 14
45130 Kouvola
☎ (+358) 5 3414 400

GB - GREAT BRITAIN

Newell Rubbermaid UK Services Ltd.
Halifax Avenue, Fradley Park
Lichfield, Staffordshire, WS13 8SS
www.nuk.co.uk

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IRL - IRELAND

Intrapharma Consumer Health
Magna Business Park,
Citywest Road
Dublin 24
United Drug House
☎ +353 1 463 2424

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UAE - UNITED ARAB EMIRATES

Arabian Ethicals Co.
AIRPORT ROAD, GARHOUH
Dubai, United Arab Emirates
☎ 04-2088400
info@arabianethicals.ae

ZA - SOUTH AFRICA

Artemis Brands (PTY) LTD
P.O. Box 130782
Bryanston 2074
☎ +2711 430 5600
www.nuk.co.za
info@artemisbrands.co.za

MAPA GmbH
Industriestraße 21-25
27404 Zeven
Germany
www.nuk.com

Matr.-Nr. 10.520.418
REV 422/02-2022